





First, select correct KidCadCrown® size from the set. As an optional step: polish the KidCadCrown® fittingly (adjust length, width, contact points by the means of a cross-cut mortising mill and subsequently smooth the crown with a rubber).

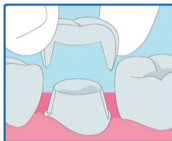


Test the KidCadCrown® in the patients' mouth.




Apply PermaCem 2.0 within the prepared restoration. (To prepare the application of the KidCadCrown®, bonding can be used as an adjuvant, as used with all common composites.)

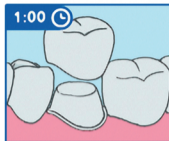


Insert the restoration after applying the PermaCem 2.0, within the 1st minute after mixing has begun.

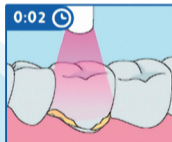


Even though PermaCem 2.0 is a dual-curing adhesive, its curing process can be accelerated by the means of an additional lightning source. Pre-cure for 2 seconds.

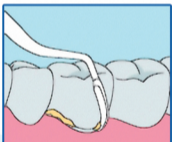

Remove all cement excess

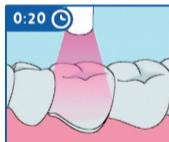

Light cure PermaCem 2.0 for 20 seconds from all sides and leave it to set for an additional 7 minutes.

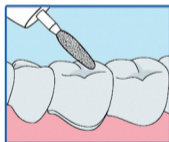

Finally, the KidCadCrown can be tailored to the opposing jaw.



Registrierungsnummer: I-1108001

Crowns' implantation using the example of DMGs' PermaCem 2.0

